



Biospecimen Exchange for Neurological Disorders

Dissecting Oligogenic Biomarkers in Ashkenazi Jews with Parkinson's Disease (AJPD)

BIOSPECIMEN COLLECTION & PROCESSING

Overview

1. Specimen uniformity and quality
2. Site Equipment
3. Procedures
 - Kit Contents and Ordering
 - Sample Labelling
 - Sample Collection and Processing
 - Shipping Samples
 - Non-Conformance
4. Contact Information

Specimen Uniformity and Quality

GENERAL REMINDERS

Specimen Standardization and Quality

Most biomarkers are sensitive to *time* and *temperature*

- Standardization of processing across sites is key
- Specimens must be processed within 2 hours of collection
- Reference the *BioSEND Biomarker Specimen Collection, Processing, and Shipment Manual* as needed
- Do not replace or supplement any kit components without first receiving approval from BioSEND/NINDS

Questions? Email biosend@iu.edu

Site Consumables and Equipment

Sites will need to supply the following items:

- Gloves
- Alcohol wipes
- Butterfly needles
- Tourniquet
- Gauze pads
- Bandages
- Microcentrifuge tube rack
- Sharps bin and lid
- Crushed ice
- Pipettes and pipette tips
- Centrifuge capable of maintaining 4°C
- -80°C Freezer
- Dry ice

Procedures

MAINTAINING SPECIMEN UNIFORMITY AND QUALITY

Biospecimen Collection Protocol

[illegible]

Kit Contents and Ordering

- All sites will be sent a Supplemental Kit with their first kit shipment
 - Contains extra blood collection tubes, processing supplies, and LP needles
 - May be used to replace items in study visit kits
- Study Visit Kits should be ordered as soon as visits are planned
 - Contains collection, processing, and shipping supplies specific to each visit
 - Includes barcoded labels
 - The supplies/labels in each study visit kit are intended for that visit only


Kit Contents and Ordering – REDCap Survey

<http://kits.iu.edu/biosend/ajpd>

Order kits online through the Kit Request Module for:

- Blood & CSF kits
- Supplemental Kit
- Extra Supplies

Please provide as much notice as possible when ordering kits and/or supplies.

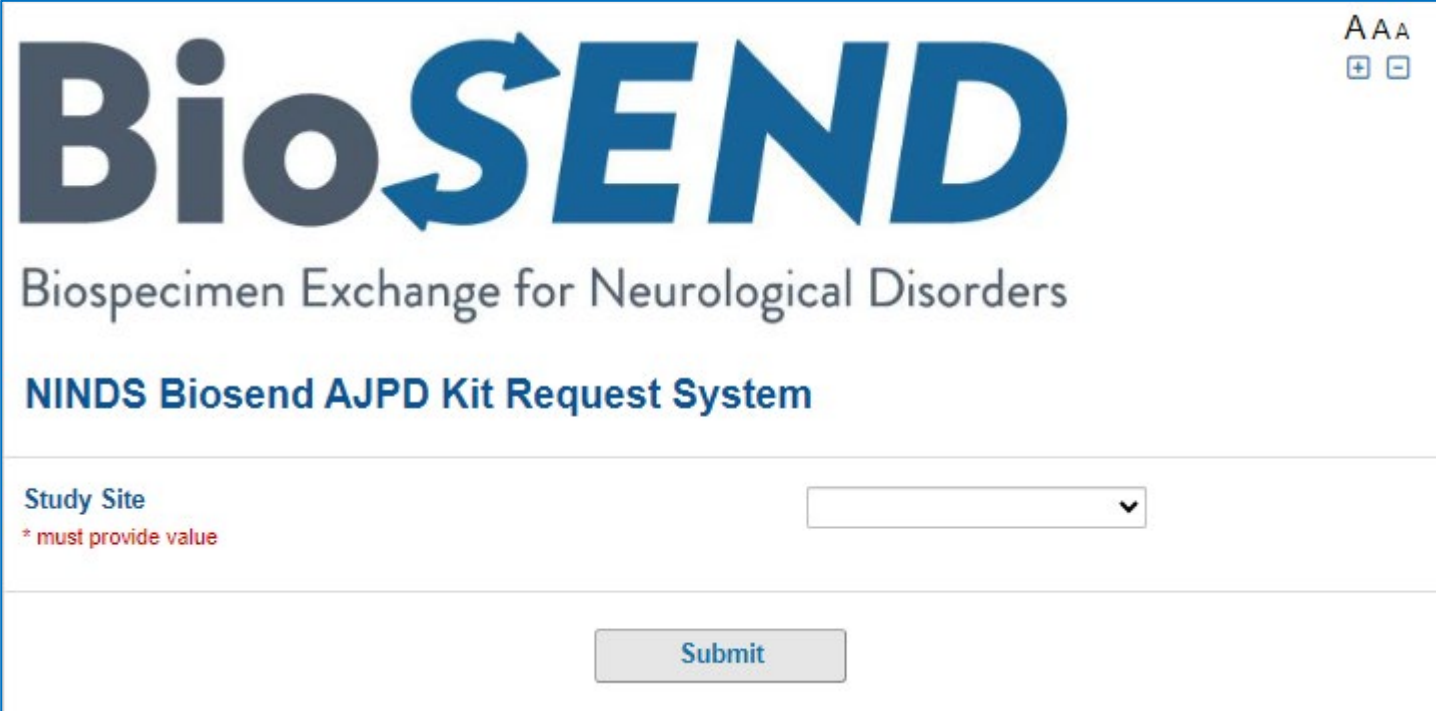

The screenshot shows the BioSEND NINDS Biosend AJPD Kit Request System interface. At the top, the logo "BioSEND" is displayed in large blue letters, with "Bio" in a standard font and "SEND" in a bold, italicized font. Below the logo, the text "Biospecimen Exchange for Neurological Disorders" is written in a smaller, grey font. Underneath that, the title "NINDS Biosend AJPD Kit Request System" is shown in a bold, dark blue font. In the top right corner, there are three small icons: "A A A" and two square buttons with "+" and "-" signs. Below the title, there is a form field labeled "Study Site" in blue text. To the right of the label is a white rectangular input box with a small downward-pointing arrow on its right side. Below the input box, the text "* must provide value" is written in red. At the bottom right of the form, there is a grey rectangular button with the word "Submit" in blue text.

Kit Contents and Ordering: Confirm Site Info

AJPD Kit Request Module

Study Site <small>* must provide value</small>	Indiana University	Select your site from the drop-down list
Indiana University School of Medicine Carolyn Dunifon Dept. of Medical & Molecular Genetics 351 West 10th Street, TK-318 Indianapolis, IN 46202-3002 (317) 274-5751 cdunifon@iu.edu		
Is the contact name above correct? <small>* must provide value</small>	<input type="radio"/> Yes <input checked="" type="radio"/> No	Verify contact information and update if needed
	reset	
New Contact Name <small>* must provide value</small>	Claire Wegel	
Is the shipping address above correct? <small>* must provide value</small>	<input type="radio"/> Yes <input type="radio"/> No	reset
Is the e-mail address above correct? <small>* must provide value</small>	<input type="radio"/> Yes <input type="radio"/> No	reset
Is the phone number above correct? <small>* must provide value</small>	<input type="radio"/> Yes <input type="radio"/> No	reset

Kit Contents and Ordering: Kit Types

AJPD Kit Request Module

- Kits are not specific to a subject or time point. After collection, sites will indicate the subject and time point to which BioSEND should link the samples.
- CSF processing kits are ordered independently of blood kits, as CSF is not required
- All specimen labels (including CSF) will be included in the Blood Collection Kit. If CSF is not collected at a visit, you may discard these extra labels

Kit Type
****Please allow two weeks for shipment****
* must provide value

☒ Blood Collection Kit

☐ Supplemental Kit

☐ CSF Collection Kit

☐ Extra Supplies

Please specify in comments if you need kits before the standard two week shipment time.

Blood Collection Kit Quantity
* must provide value

Comments

Expand

Kit Contents and Ordering: Kit Breakdown

AJPD Kit Request Module

Comments

Expand

Each Baseline and Longitudinal Visit Collection Kit

Blood Collection Kit Contents:

2 - Lavender-top EDTA tube (10 ml), glass

2- Purple-top EDTA tubes (3 ml), plastic

2 - Red-top serum tube (10 ml), glass

2 - PAXgene® tube (2.5 ml)

15 - Siliconized cryovial (2ml)

1- Urine Cup

4 - 15ml Centrifuge tube, individually (Blue) wrapped

4 - Disposable transfer pipette (3ml)

1 - 25-slot Cryobox

1 - Shipping label packet (Dry Ice, Fragile, UN3373)

1 - Airway bill envelope

1 - Shipping container for dry ice shipments

2 - Biohazard bag with absorbent sheet


8 - Individual tube bubble pouch

30 - Cryohold pre-printed labels-- ST Numbers

Submit


Kit contents of selected kit will appear at the bottom of the page

Kit Contents and Ordering: Blood Kits

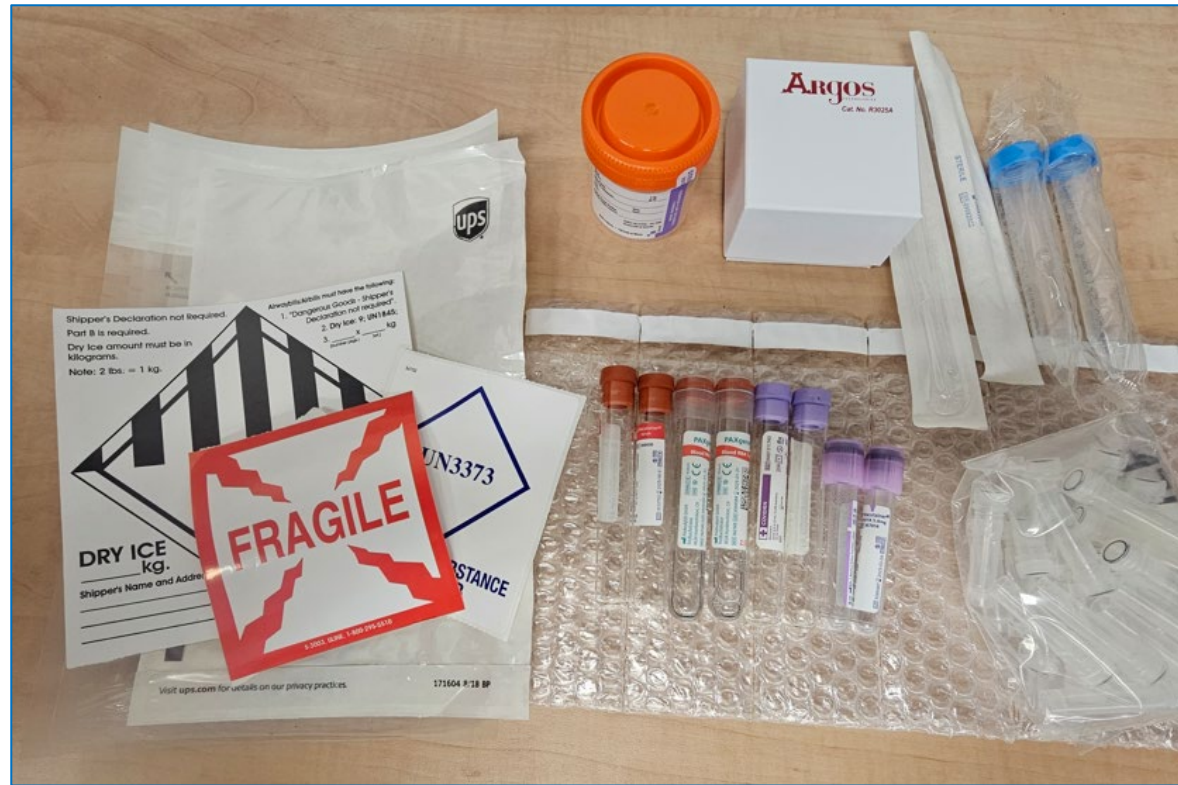

Kit Contents and Ordering: CSF Kits

CSF:

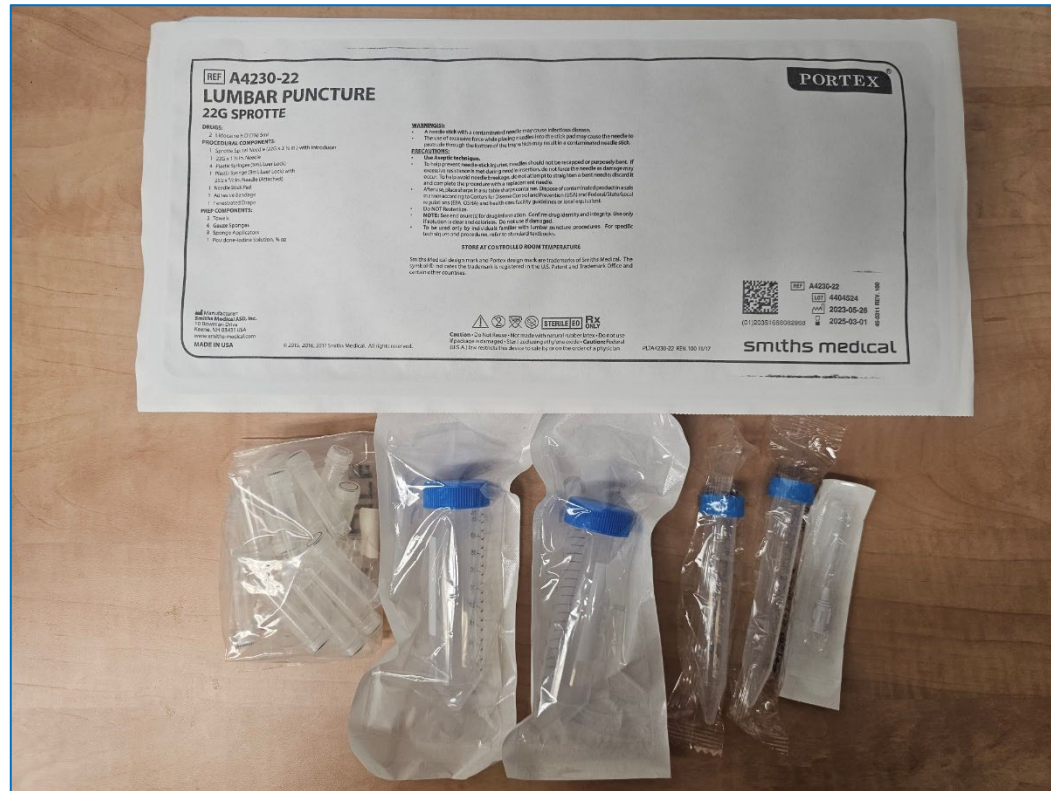

LP Tray:



Collection Volumes

Total blood, urine and CSF volumes

Sample Type	Amount
Whole Blood for RNA	5 ml
Whole Blood for Plasma and Buffy Coat	20 ml
Whole Blood for Serum	20 ml
Whole Blood for Banking	6 ml
Urine	20 ml
Cerebrospinal Fluid	10 ml


Kit and Supply Ordering

- Click “Submit” to send order to BioSEND; staff will confirm receipt of your order
- Please allow two week turn-around time for kit shipments
- If urgent request needed, please note date needed by in comments and email BioSEND. We cannot guarantee urgent orders, but we will do our best to accommodate.
- BioSEND will send confirmation of shipment and tracking number when supplies ship

Sample Labelling: Example Labels

Labels are provided by Indiana University


- Please check that all samples are properly labelled to ensure correct identification by IU
- If do not have enough labels to complete a visit, please contact IU *immediately*
- Labelling the tubes during processing prevents sample mix-ups

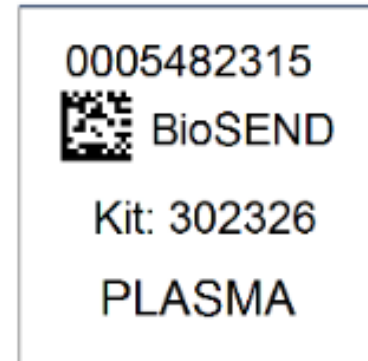

Sample Labelling: Label Placement


Please...

- Label all collection and aliquot tubes before cooling, collecting, processing, or freezing samples
- Label only 1 subject's tubes at a time to avoid mix-ups
- Wrap the label around the tube horizontally - label position is important for all tube types
- Make sure the label is completely adhered by rolling between your fingers

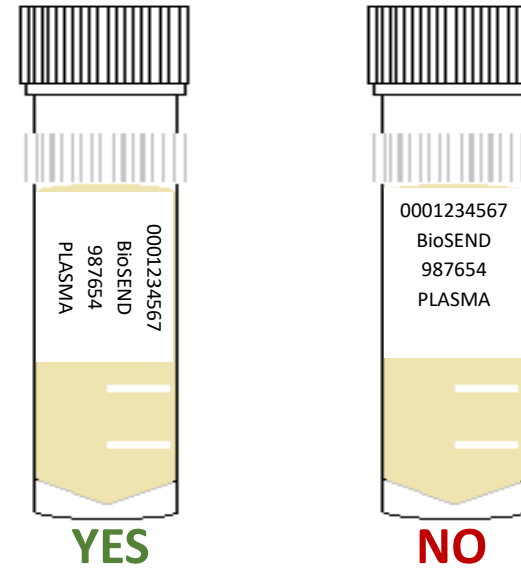


Sample Collection and Processing


Blood Tube Draw Order

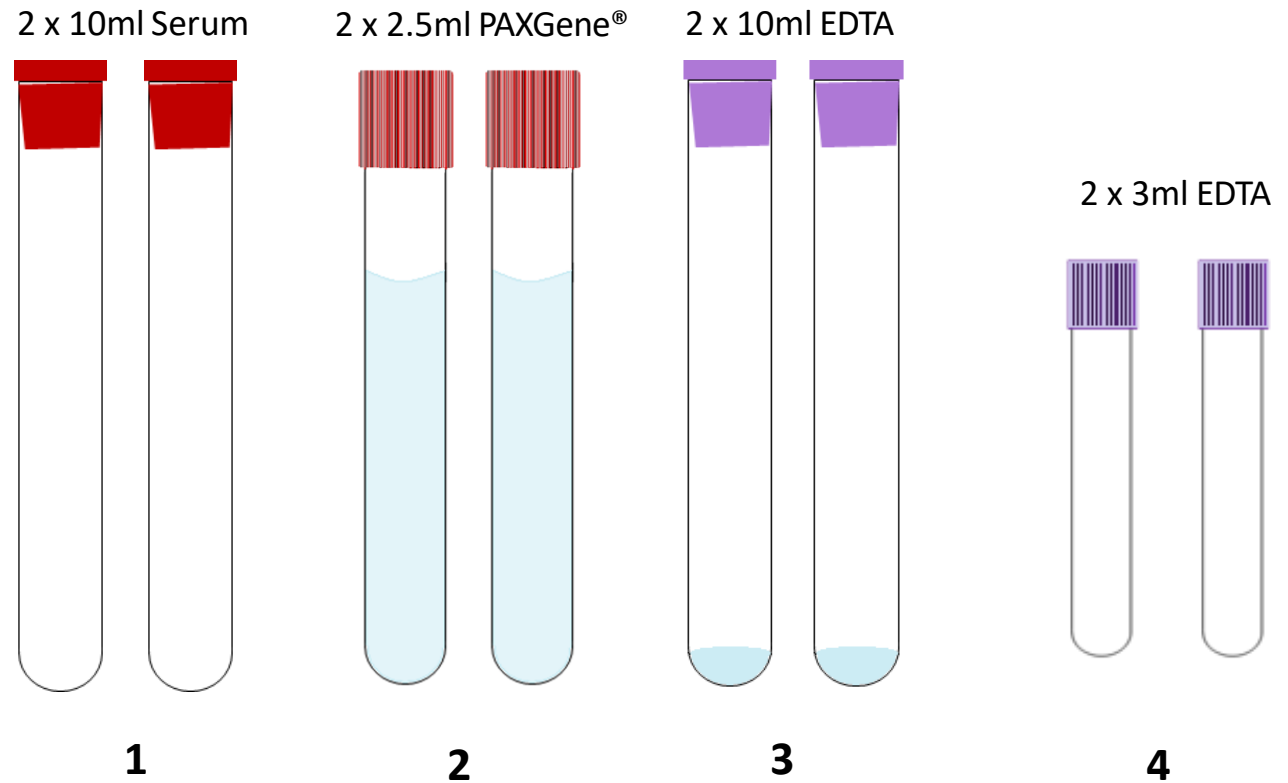


Sample Collection and Processing: Serum

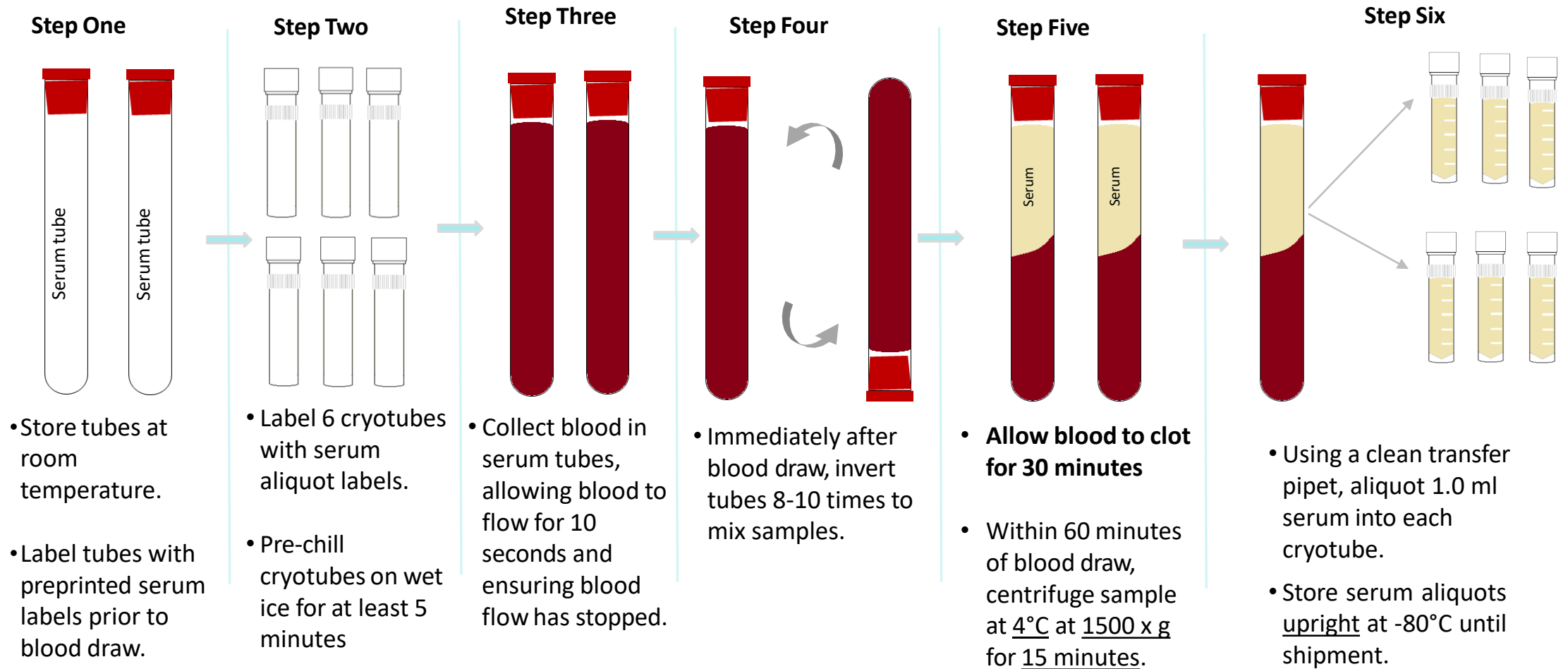


Sample Collection and Processing: Whole blood RNA

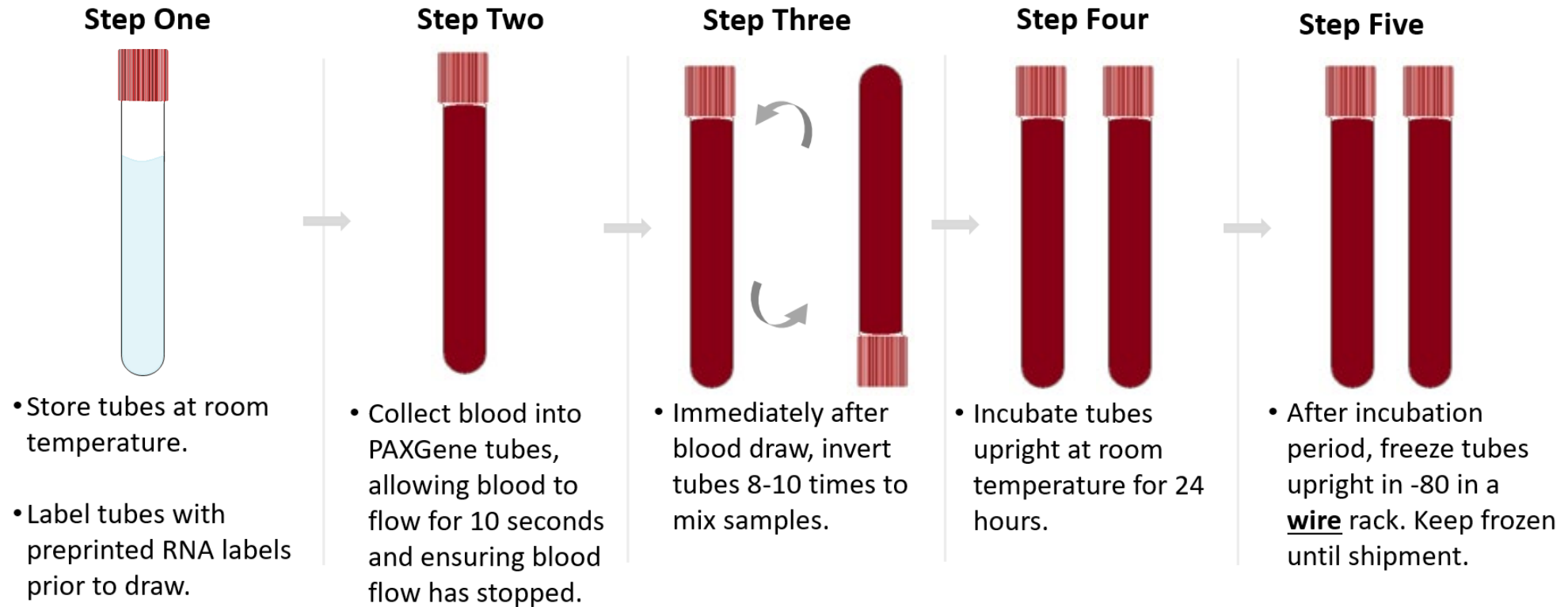


Sample Collection and Processing: Plasma & Buffy Coat

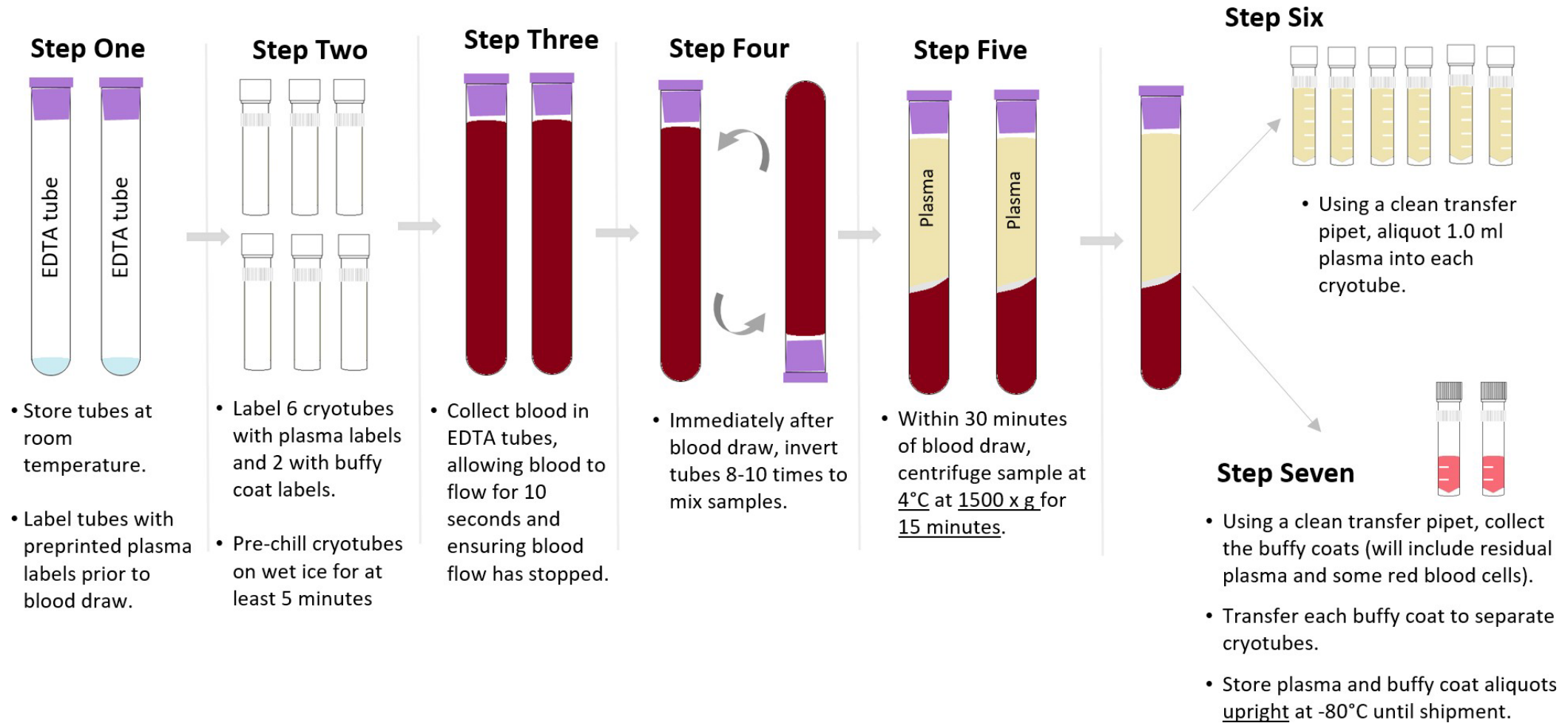

Sample Collection and Processing: Whole Blood

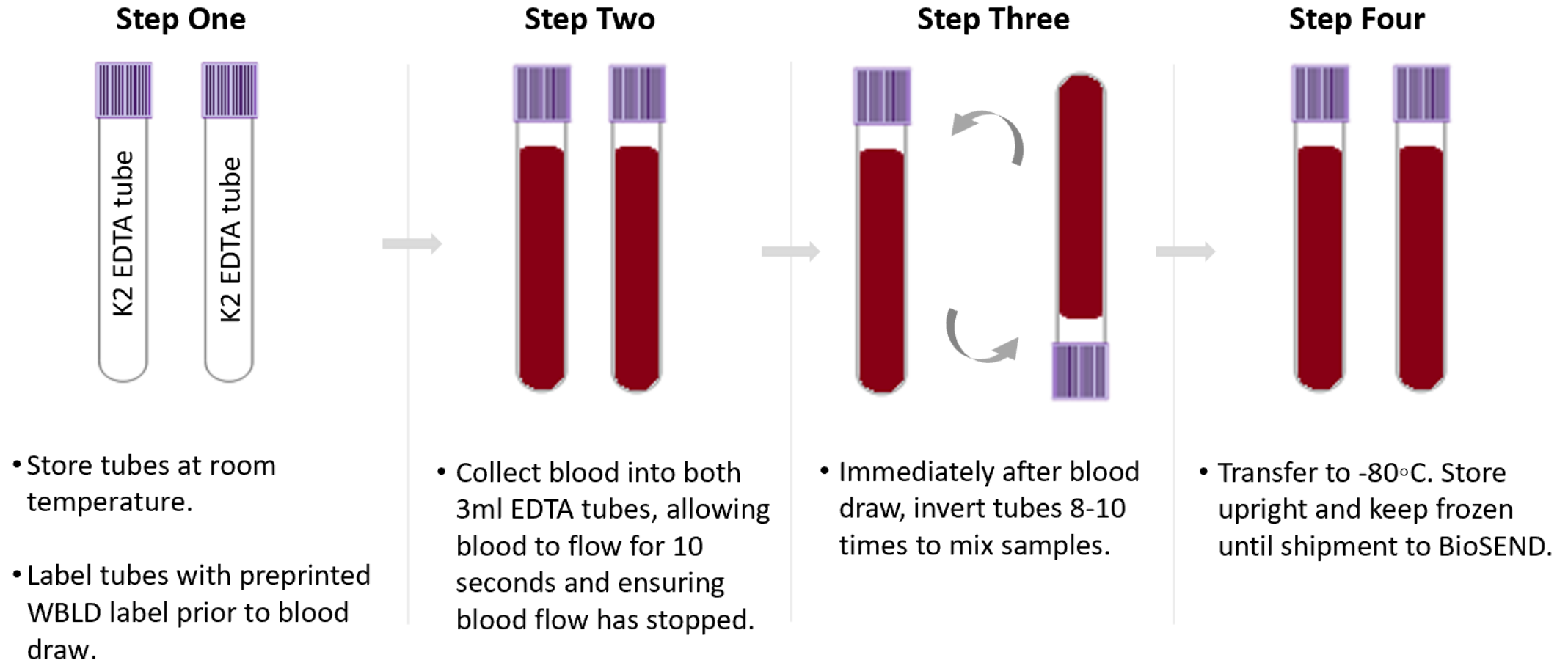

Sample Collection and Processing: CSF

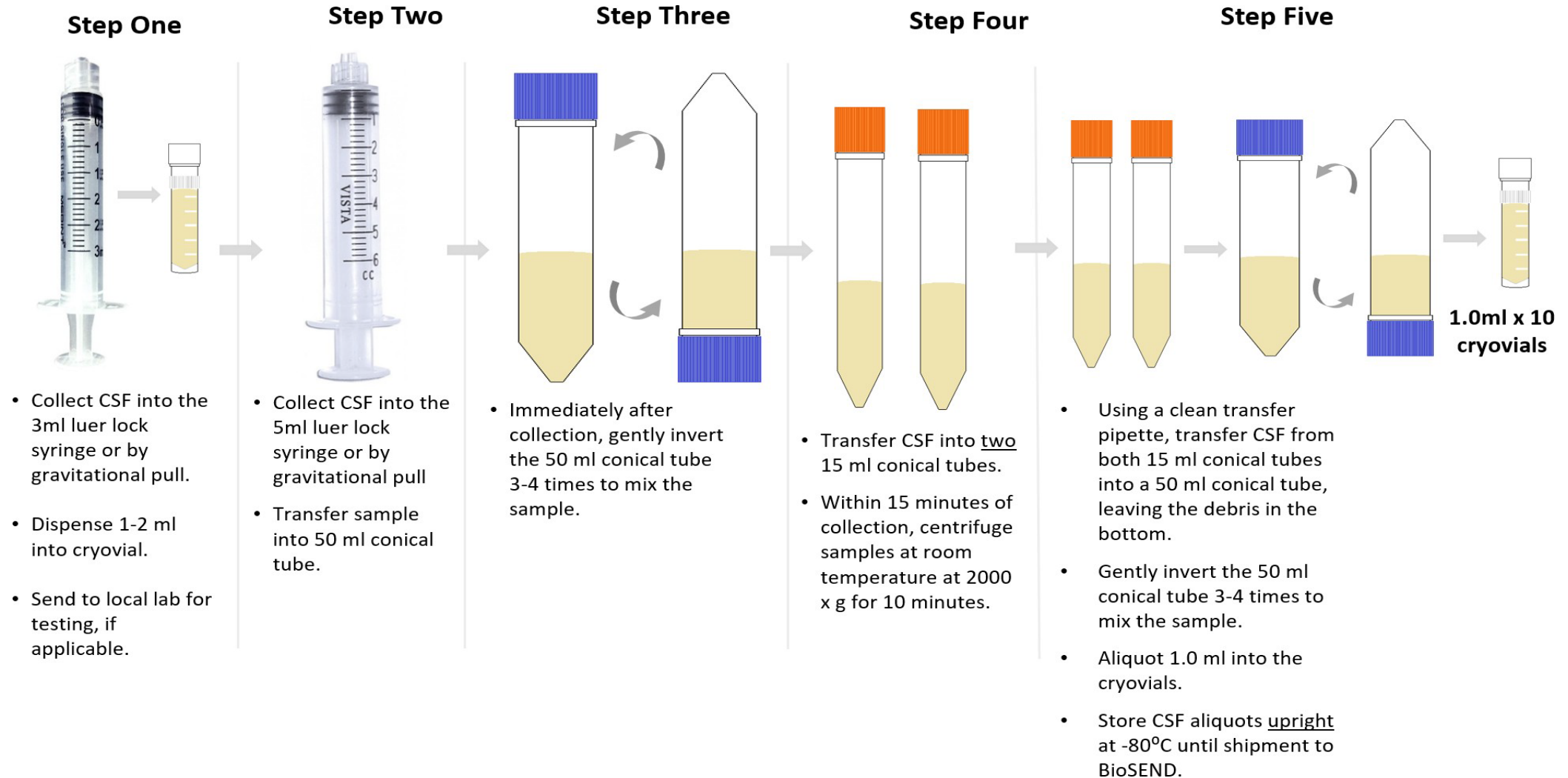

Urine Sample Collection


1. Label one urine collection cup prior to urine collection with a pre-printed “URINE” label.
2. Ask study subject to collect a urine specimen in the collection cup. Urine should be collected midstream and should remain as sterile as possible.
3. Label two 15 ml conical tubes with pre-printed “URINE” labels.
4. Transfer 10mL urine from the collection cup into each of the two 15 mL conical tubes.
5. Within 60 minutes of collection, freeze and store samples **upright** at -80°C until shipment.



Sample Collection and Processing: Aliquots

Filling biomarker serum, plasma, and CSF aliquots:

- Fill as many cryovials as possible to 1.0 ml (plasma, serum & CSF)
- Over-filled vials may burst in freezer!
- Ship ALL material to IU, even if final vial is less than standard volume

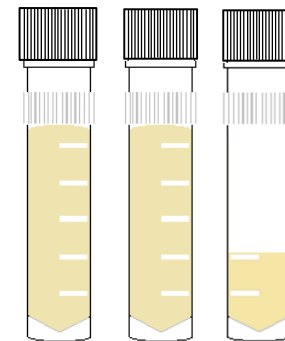


YES



NO

Sample Collection and Processing: Timeline

Timeline for blood processing

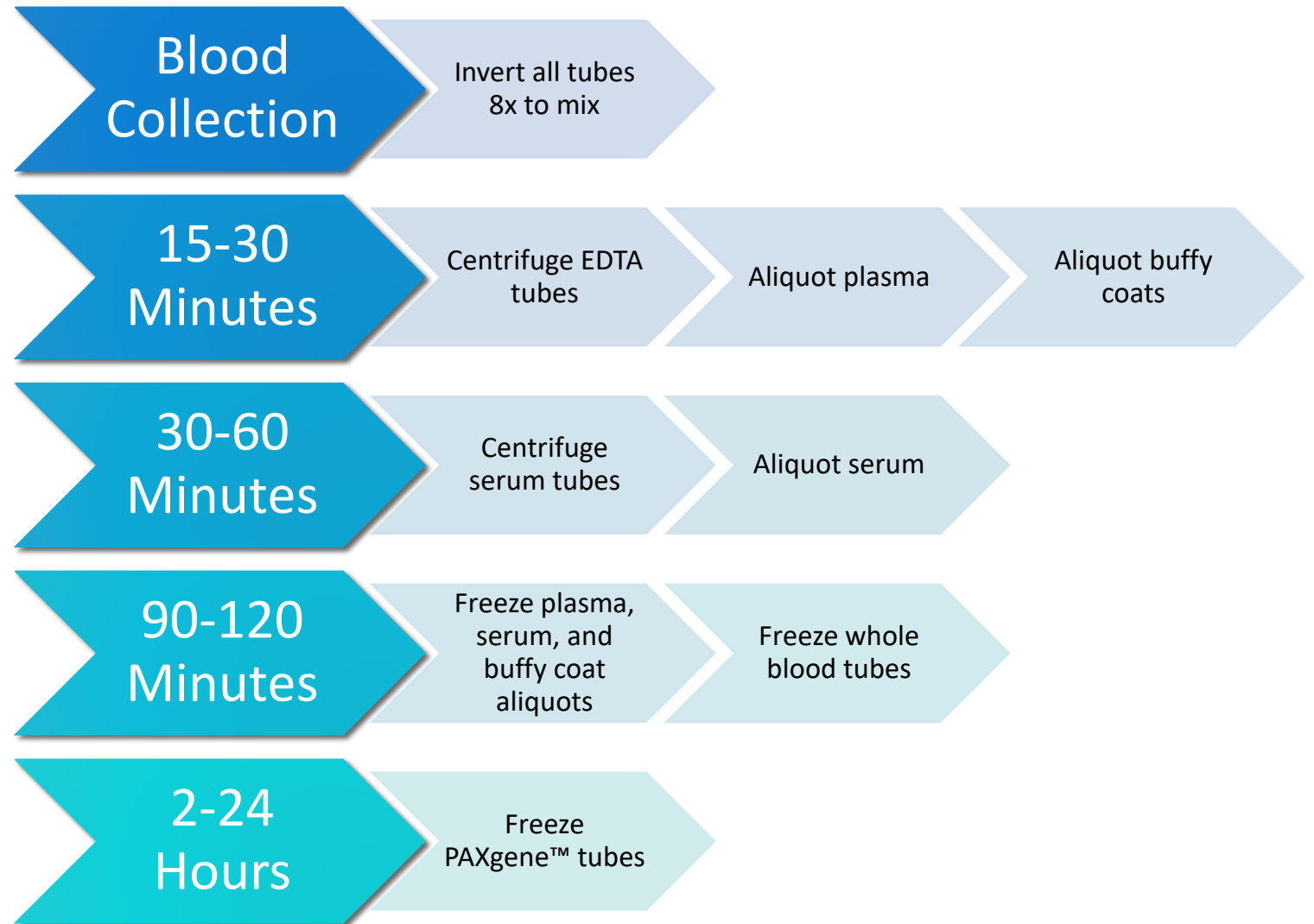

Sample Collection and Processing: Issue #1

Troubleshooting Blood Collection

Issue #1: Tube with little/no vacuum

- Always check expiration date on the tube before beginning blood draw and discard expired tubes
- Store tubes at “room temperature” – extreme temperature can affect vacuum
- Keep extra vacutainer tubes from supplemental kit nearby during blood draw to replace “bad” tubes
- If this is a frequent occurrence, report tube type and lot number to IU.

Sample Collection and Processing: Issue #2

Troubleshooting Blood Collection

Issue #2: Hemolyzed serum and/or plasma caused by incorrect collection

Cause: Blood Collection Methods	Corrective Action
Improper venipuncture site	Draw from median cubital, basilic, and cephalic veins from antecubital region of arm
Prolonged tourniquet use	Tourniquet should be released after no more than 1 min, excessive fist clenching should be avoided
Not allowing alcohol to dry on skin before venipuncture	Without touching, allow the venipuncture site to air dry
Use of too large/small bore needle resulting in excess force applied to blood	Avoid using too small/large needle. Needle size dependent on the subject's physical characteristics & amount of blood to be drawn. Most commonly used sizes are 19 – 23.
Pulling/pushing plunger too fast while drawing/transferring blood	Avoid drawing the syringe plunger too forcefully when collecting blood
Ensure all blood collection assemblies are fitted securely, to avoid frothing	

For more information, visit: http://www.bd.com/vacutainer/pdfs/techtalk/TechTalk_Jan2004_VS7167.pdf

Sample Collection and Processing: Issue #2 continued

Troubleshooting Blood Collection

Issue #2: Hemolyzed serum and/or plasma caused by incorrect processing

Cause: Sample Processing Methods	Corrective Actions
Vigorous mixing/shaking	Gently invert blood collection tube when mixing additive with specimen, follow guidelines in Biologics Manual regarding number of times to invert each type of tube
Not allowing serum to clot for recommended time	Serum tubes without clot activator should be allowed to clot for 60 min in a vertical position
Exposure to excessive heat or cold	Keep samples at ambient temperature until processing
Prolonged contact of serum/plasma with cells	Do not store uncentrifuged samples beyond recommended time

For more information, visit: http://www.bd.com/vacutainer/pdfs/techtalk/TechTalk_Jan2004_VS7167.pdf

Sample Collection and Processing Form

Direct link:

<https://redcap.link/AJPDSampleForm>

First part captures basic subject and visit information

Returning?

AA

BioSEND

Biospecimen Exchange for Neurological Disorders

Please verify/update the information below. When you click the "Submit" button below, a PDF copy of the Sample Record and Shipment Notification Form will be emailed to you.

Please print a copy of that document and **include it in the shipping container.**

BioSEND AJPD Study

Study Site

Email address of staff member completing this form

Note: A copy of the completed sample form and the shipping manifest will be sent to this address.

GUID:

IU Kit Number

Sex (used for DNA quality control)

Visit

Date of venipuncture blood collection

Today

M-D-Y

Time of venipuncture blood collection

Now

H:M

Use 24 Hour clock

Date participant last ate

Today

M-D-Y

Time participant last ate

Now



H:M

Sample Collection and Processing Form

Direct link:

<https://redcap.link/AJPDSampleForm>

Second part captures collection information

RNA Paxgene	
Number of PAXGene™ tubes shipped:	<input type="text"/>
SERUM	
Number of SERUM aliquots shipped:	<input type="text"/>
Each aliquot should be 1.0 mL	
PLASMA EDTA	
Number of PLASMA EDTA aliquots shipped:	<input type="text"/>
Each aliquot should be 1.0 mL	
Number of BUFFY COAT aliquots shipped:	<input type="text"/>
WHOLE BLOOD EDTA	
Number of WBLD tubes shipped:	<input type="text"/>
Two 3ml EDTA tubes expected	
URINE	
Date of urine collection	<input type="text"/>  <input type="button" value="Today"/> M-D-Y
Time of urine collection	<input type="text"/>  <input type="button" value="Now"/> H:M
Use 24 Hour clock	
Number of URINE tubes shipped:	<input type="text"/>
Two tubes expected	
CSF	
Was CSF collected at this visit?	<input type="button" value="Yes"/>
	<input type="button" value="No"/>
<input type="button" value="reset"/>	
NOTES	
Please record any issues with collection/processing:	<input type="text"/>

Sample Collection and Processing Form

Direct link:

<https://redcap.link/AJPDSampleForm>

PDF form of responses will be emailed to you. **Print a copy of the Frozen Shipping Manifest and include with shipment.**

AJPD Sample Record and Shipment Notification Form

Page 1

Please verify/update the information below. When you click the "Submit" button below, a PDF copy of the Sample Record and Shipment Notification Form will be emailed to you.

Please print a copy of that document and include it in the shipping container.

BioSEND AJPD Study

Study Site

☐ Mount Sinai

Email address of staff member completing this form

Note: A copy of the completed sample form and the shipping manifest will be sent to this address.

GUID:

IU Kit Number

Sex (used for DNA quality control)

☐ Male
☐ Female
☐ Other

Visit

☐ BL
☐ 6M
☐ 12M
☐ 18M
☐ 24M
☐ 30M
☐ 36M
☐ 42M
☐ 48M
☐ 54M

Date of venipuncture blood collection

Time of venipuncture blood collection

(Use 24 Hour clock)

Date participant last ate

Time participant last ate

Sample Collection and Processing Form

Submission of the Sample Form serves as shipment notification to BioSEND

- Must be completed prior to shipment
- If samples are shipped and not received, BioSEND will follow-up with courier. It is recommended that sites also track shipment to ensure safe delivery

Shipping Frozen Samples: Tips

Packing and Shipping Frozen Samples

- All samples ship frozen on dry ice
- Frozen samples should be shipped **only** Monday through Wednesday
- Always fill carton to **top** with dry ice
- Do not pack shipment until the day of pickup



Shipping Samples

Packing and Shipping Frozen Samples

Class 9 Dry Ice Label should not be covered with other stickers and must be completed, or UPS will reject/return your package!

Shipper's Declaration not Required.

Dry Ice amount must be in kilograms.

Note: 2 lbs. = 1 kg.

Airwaybills / airbills must have the following:
1. Dry Ice; 9; UN 1845
2. $\frac{\text{Number}}{\text{(Number pkgs)}} \times \frac{\text{wt}}{\text{(wt)}} \text{ Kg}$

Net weight of dry ice in kg

Dry Ice kg.

Your name & address

Shipper's Name and Address

UN 1845

Consignee Name and Address

IU information and address

9

06426 1/01 RRD

Shipping Samples: Frozen

Do not ship more than 4 biohazard bags in a single shipper (equivalent to two subject-visits).

- Allows room for dry ice to keep samples frozen in transit
- Minimizes loss in the rare but unfortunate event of courier issues



Shipping Samples – UPS: <https://kits.iu.edu/UPS>

The screenshot displays the 'SHIP FROM' and 'SHIPMENT INFORMATION' sections of the IU School of Medicine shipping form. Annotations with arrows point to specific fields and buttons:

- Search for address:** Points to the search icon in the 'Ship From' section.
- Choose Study:** Points to the 'Study Group' dropdown menu in the 'Shipment Information' section.
- Enter weight:** Points to the 'Weight' and 'Dry Ice Weight' input fields in the 'Shipment Information' section.
- Schedule Pickup:** Points to the 'Pickup Request' button in the 'Shipment Information' section.
- Click "Ship":** Points to the 'Ship' button in the bottom right corner.

The form contains the following data:

Ship From	
Code	
Company	Icahn School of Medicine - Mt. Sinai
Contact	Kenny Persaud
Address 1	1425 Madison (Icahn Building)
Address 2	Icahn L4 39
Address 3	
City	New York
State/Province	NY
Postal Code	10029
Country/Territory	United States

Shipment Information	
Study Group	SSBC
Weight	20 LB
Dry Ice Weight	10 LB
Description of Return	Biologic Specimens

Buttons: **Pickup Request**, **Reset**, **Ship**

Shipping Samples via UPS

IU UPS ShipExec Shipping Portal

- Print out UPS air waybill
- Ensure all elements (barcode, return address, etc.) printed clearly
- Fold and insert UPS air waybill into clear plastic sleeve on package

JOHN SMITH INDIANA UNIVERSITY 410 WEST 10TH STREET INDIANAPOLIS IN 46202	2 LBS	1 OF 1
RS		
SHIP TO: SCHOOL OF MEDICINE 317-278-2694 INDIANA UNIVERSITY TK 217 351 W 10TH ST INDIANAPOLIS IN 46202		
	IN 461 9-01	
		
UPS NEXT DAY AIR		1
TRACKING #: 1Z 976 R8W 84 3985 8595		
 SAMPLE		
BILLING: P/P DESC: Biological Specimens RETURN SERVICE		
Reference No.1: 4087277		
XOL 20.03.09 NV45 83.0A 12/2019		 TM

Shipping Samples: Closures

Date	Holiday
January 1	New Year's Day
3 rd Monday in January	Martin Luther King, Jr Day
4 th Monday in May	Memorial Day
June 19	Juneteenth (observed)
July 4	Independence Day (observed)
1 st Monday in September	Labor Day
4 th Thursday in November	Thanksgiving
4 th Friday in November	Friday after Thanksgiving
December 25	Christmas

Please also consider weather when shipping. UPS will post service updates on their webpage. Reach out to BioSEND if you are unsure if it is safe to ship.

Non-Conformance

Non-conformance to standard procedures may reduce the utility of the biospecimens:

- Not processing serum/plasma within 2 hours of collection allows for breakdown of certain proteins and small molecules
- Over/under centrifuging changes specimen composition



Non-Conformance Reporting con't

Most common non-conformance issues:

- Shipment notification not sent
- Samples shipped for weekend/holiday delivery
- Sample form incomplete/inaccurate
- Low volume
- Unlabeled or mislabeled tube(s)
- Sample hemolysis



Non-Conformance and Inventory Reporting

Most common non-conformance issues:

- BioSEND will notify sites directly of any issues upon receipt
- BioSEND will email sites a monthly inventory report of all samples received from that site to date
- ***If you are experiencing issues, please reach out to us for help! It is much easier to prevent an issue before sample collection & shipment than trying to fix it after the fact***

Contacts

Indiana University

General Questions/Shipment Notifications:

biosend@iu.edu

317-278-6158

Request kits:

<http://kits.iu.edu/biosend/ajpd>